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FIM Medical would like to thank you for purchasing the **Audiolyser® ADL20** audiometer and we hope you are completely satisfied with the product.

Equipment included in the packaging

- Audiolyser® ADL20 device with headphones and USB lead (in the cover)
- CD containing user manual and Audiowin®20 software

The user manual is available on the CD or in the Audiowin20® software "Help" Tab. The User Manual can also be downloaded from our website www.fim-medical.com in the "Documentation" and the Audiowin20® software in "Downloads". To download the software, type in the password labelled on the CD sleeve.

*Finally, if you prefer a User Manual in paper format, please contact us by email at contact@fim-medical.com or in writing at **FIM MEDICAL – 51 rue Antoine Primat - 69100 Villeurbanne – France**. A manual will be sent to you free of charge within approximately 7 days.*

- Information sheet
- Calibration certificate
- Transport cover

Safeguard the device original packaging and padding.

Caution For annual maintenance and After Sales Service, the device must be returned in its original packaging.

Compulsory use of FIM Medical hygienic single-use earpad covers ø11cm (1 box of 100 earpad covers supplied with your Audiolyser® ADL20)

The hygienic single-use earpad covers must systematically be used for each test, and changed after each patient.

Caution For hygiene and biocompatibility reasons, it is compulsory to use the FIM MEDICAL brand of single-use earpad covers.

These earpad covers have been developed specifically by FIM Medical to meet ISO 10993 material biocompatibility requirements, as well as guaranteeing perfect sound transmission whilst respecting IEC 60645-1.

REACH (EC) n°1907/2006 article 33: some internal fastening components of the Holmco PD-81 headset of the Audiolyser® ADL20 include more than 0.1% by mass of Lead (CAS 7439-92-1) and of Imidazolidine-2-thione (CAS 96-45-7). These components are isolated and secured from any contact with the user.

Nb. Lead, Imidazolidine-2-thione are substances considered to be of very high concern.