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FIM Medical would like to thank you for purchasing the **Tabataba® V3** carbon monoxide breath analyser and we hope you are completely satisfied with the product.

**Equipment included in the transport cover:**

- Tabataba® V3 device
- Power supply and USB-C cable for charging the battery
- User manual
- Information sheet
- Calibration certificate

**Device power On/Off: Press and hold the power button for 2 seconds.**

**Without annual calibration, measurement drift may compromise accuracy of results.**

**Warning Please make sure to keep the original packaging for shipments to maintenance.**

**Do not immerse or spray the device with liquid for disinfection purposes.**

**Wi-Fi Connection and Web App Access:**

You can manage the device and view saved data from a PC or smartphone through Wi-Fi access.

The Wi-Fi access is named according to the serial number of your device. Example:

|                    |                 |
|--------------------|-----------------|
| Serial number:     | 230001          |
| Wifi network name: | Tabataba-230001 |
| Wifi password:     | TT230001!       |

Webapp access address: <http://tabataba.local> or <http://192.168.4.1>

Default login credentials:

| Username | Password |
|----------|----------|
| user     | user     |
| admin    | admin    |

**Warning To ensure the protection of data stored on Tabataba® V3, it is imperative to change the default passwords and regularly update passwords for different users.**

**Touchscreen Usage:**

Tabataba® V3 features a resistive touch screen.

Unlike capacitive touch screens found in smartphones, the resistive touch screen requires firmer pressure. Avoid light gestures or taps, as they may not register. For optimal and precise usage, apply pressure with the fingertip or nail.

You can also use a stylus or pen tip.