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
Instruction sheet

Pipette and Trapper for TABATABA®

FIM MEDICAL pipettes and trappers are accessories developed specifically for the TABATABA® device (analyser of carbon monoxide (CO) in expired air).

Caution: U.S.A. Federal Law restricts this device to sale by or on the order of a physician.

Caution: Always use a pipette and trapper with the TABATABA®.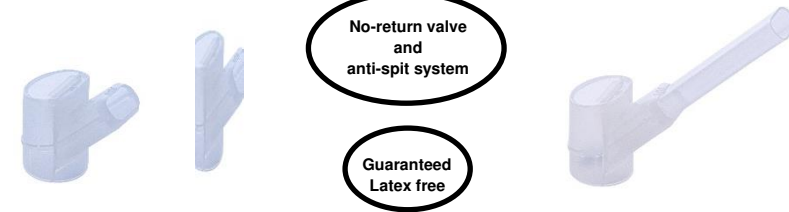

Installation:

- Remove the trapper and pipette from the packaging
- Do not touch the mouthpiece of the pipette with the fingers
- Insert the pipette into the trapper and connect the set to the device

Use:

- Systematically change the single use pipette between patients
- Systematically change the single use trapper between patients
- **Caution:** risk of cross contamination if disposable consumables are reused
- With a slightly damp cloth, clean the CO detector with a non-alcohol bactericide fungicide solution
- Do not breathe into the pipette (risk of contamination)

Disposal:

Used or dirty accessories must be disposed of with biological waste (DASRI).

Features:

Pipeta	⊗ Single use
Trapper	⊗ Single use
Packaging	Compartmentalized sachets non sterile
Expiry date	See cardboard box label
Storage / transport	T°: 0°C - 50°C Humidity: 75% max.
Medical class	Class I

First marketed: 2007

Labels:

CE Marking RDM 2017/745

Manufacturer identification

Date of manufacture

Consult the accompanying documents

Lot number

Expiry date

Humidity Level Limit

Single use

Storage temperature: between 0 and 50°C