

Notice d'utilisation Pince-nez

Le pince-nez FIM MEDICAL est un accessoire développé spécifiquement pour les spiromètres SPIROLYSER® SPL10® USB et SPIROLYSER® Q13®.

Attention : Utilisation uniquement par un médecin ou un professionnel de santé.

Attention : Ne jamais confier le pince-nez aux enfants.

Attention : Le pince-nez est à usage unique.



Installation :

- Sortir le pince-nez de son emballage
- Ne pas toucher la partie en mousse avec les doigts mais le tenir par le corps
- Positionner le pince-nez sur votre nez afin de boucher les narines


Utilisation :

- Changer systématiquement le pince-nez à usage unique entre chaque patient

Élimination des déchets :

- Les accessoires souillés doivent faire partie d'une collecte sélective pour déchets biologiquement souillés (DASRI).

Caractéristiques :

Pince-nez avec mousses biocompatibles	 Usage unique
Date limite d'utilisation	Voir étiquetage du carton
Stockage / transport	T° : entre 0°C et 50°C Humidité : 75% max.
Classe médicale	Classe I

Symboles :



Marquage CE MDR 2017/745



Identification du fabricant



N° de lot



Date limite d'utilisation



Date de fabrication



Usage unique

Instructions Nose clip

The FIM MEDICAL nose clip is an accessory developed specifically for SPIROLYSER SPL10® USB and SPIROLYSER® Q13® spirometer.

Caution: U.S.A. Federal Law restricts this device to sale by or on the order of a physician.

Caution: Keep out of reach of children.

Caution: The nose clip is single-use.



Installation

- Remove nose clip from the packaging
- Do not touch the foam pads
- Place the nose clip on the nose so as to block the nostrils


Use

- Systematically change the single-use nose clip between patients

Disposal

- Used or dirty accessories must be disposed of with biological waste (DASRI).

Features

Nose clip with biocompatible foam pad	 Single-use
Expiry date	See cardboard box label
Storage/transport	T°: 0°C - 50°C Humidity: 75% max.
Medical class	Class I

Symbols:



CE Marking MDR 2017/745



Manufacturer identification



Consult the accompanying documents



Lot number



Single use



Expiry date



Manufacturing date