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Public Health England
Porton Down
Salisbury
Wiltshire, SP4 0JG
United Kingdom

The *Ergofilter® SP1* (with anatomical mouthpiece) and *SP1M* (with flexible anatomical jaw-piece) are bacterial and viral filters for use with the FIM MEDICAL SPL10 USB spirometer.

Ergofilter® filters are designed and adapted for lung function tests (spirometry). Ergofilter® Filters are designed so that their combined resistance to the spirometer SPL10 USB does not exceed the ATS / ERS recommendations.

Ergofilter® technical features:

Filter type	 Single use
Packaging	Individual sachets non sterile
Expiry date	See cardboard box label
Storage / transport	T°: 0°C - 50°C Humidity: 75% max.
Membrane type	Electrostatic
Bacterial retention (aerosolization of microorganisms <i>Bacillus atropheus</i> NCTC 10073)	99,99% ⁽¹⁾
Viral retention (aerosolization of microorganisms <i>MS-2 coliphage</i> NCIMB 10108)	99,84% ⁽¹⁾
Resistance to 14L.s ⁻¹	1.1 cmH ₂ O.L ⁻¹ .s
Dead volume	65ml
Medical class	Class IIa

Labels:

CE marking Directive 93/42/EEC
+ N° of notified body

Consult the accompanying documents

Single use

Manufacturer identification


Batch number

Expiry date

Recommendations for use:

- Remove the filter from its sachet
- Do not touch the mouthpiece of the filter with your fingers
- Use the filter the correct way; it is not reversible
- Connect the cone on the spirometer ("mouthpiece" side), then connect the filter to the cone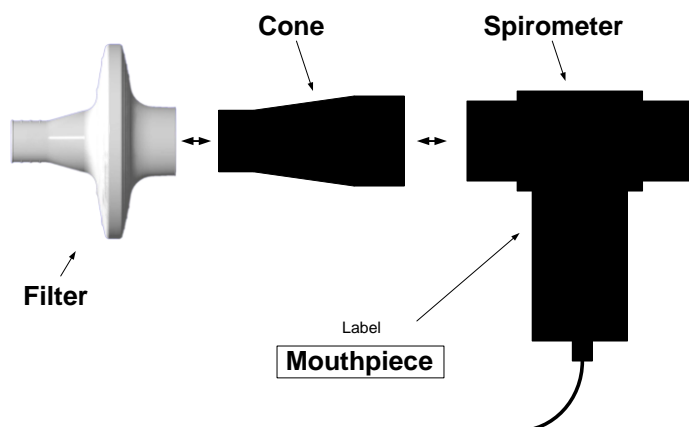

Caution: Always use an *Ergofilter® SP1* or *SP1M* filter between the patient and the spirometer.

Caution: *Ergofilter® SP1* and *SP1M* filters are single use.

Caution: The use of filters does not exempt equipment from regular disinfection.

Caution: Used or dirty accessories must be disposed of with biological waste (DASRI).

⁽¹⁾ Bacterial and virale efficacy trials on the *Ergofilter®* filters were performed by the laboratory:

Public Health England
Porton Down
Salisbury
Wiltshire, SP4 0JG
United Kingdom