

CO-TESTER



CATCH
YOUR BREATH

TABATABAv3[®]



**EXHALED AIR
CO ANALYSER:**

- ▶ Modern and simple interface
- ▶ Large touchscreen
- ▶ Instant results in PPM,
% HbCo and % fetal HbCo
- ▶ Cell life: 5 years



CE
0459



USB-C
charging port
Battery-free operation

Switch on button



Large 2.8" touchscreen

Disposable Trapper & Pipette

with non-return valve and single-use mouth piece
(compatible with Tabataba V2 and V3).

YOUR NEW ALLY AGAINST ACTIVE AND PASSIVE SMOKING

Easy to use, accurate and reproducible, the **TABATABA v3®** is a measurement device intended for health professionals (tobaccologist, occupational health worker, associations, maternity departments, lung specialist, chemist, etc.)

It measures the level of **CO** in expired air and provides an **accompaniment** for **stop smoking programmes**. It can also measure **CO levels in ambient air**. Widely used by doctors, it is also an excellent teaching instrument, its power of awareness to stop smoking smoking indisputable.

HAND-CONTROL YOUR DEVICE

With its large touchscreen, the **TABATABA V3** offers an all-new user experience.

Computerised, battery-less and **cordless**, it offers greater freedom. The battery can be recharged using the USB cable. **Instant** consultation of **Information and results** on the device and/or **exported by wifi**.

TABATABAv3®

WEB APP ADVANTAGES

- ▶ Product information management (serial number, calibration date, etc.)
- ▶ Management of secure patient information (recorded measurements, patient identity, etc.)
- ▶ Device settings (expiration time, etc.)
- ▶ Export data in CSV
- ▶ Generate a patient report (PDF)



STEP 1



Press
«Start test»

STEP 2



The patient must inhale,
then hold their breath
during the countdown

STEP 3



The patient must
blow slowly until
the signal disappears

STEP 4



Result displayed
in PPM*, HbCO%*
or HbCOf%*

STEP 5



Save data

*PPM: CO particles per million air particles. % HbCO: carboxyhaemoglobin level %HbCOf: fetal carboxyhaemoglobin level

YOUR DISTRIBUTOR

**SMOKING
KILLS
...SO DOES
DOING
NOTHING!**

Get into the habit of having your
«**CO calculator**» to hand.
The **TABATABA v3®** quickly and reliably
tests levels of expired CO in active and
passive smoking.



ESSENTIAL ACCESSORIES

Not sold with device:

- Single-use pipette (pack of 250)
- Single-use trapper (pack of 250)



51 rue Antoine Primat - CS 60194
69625 Villeurbanne Cedex France

Tél +33 (0)4 72 34 89 89
Fax +33 (0)4 72 33 43 51
contact@fim-medical.com

CE
0459